

Instructions for
cleaning and
disinfecting your
PARI nebulizer



Step #1 Clean the nebulizer.

Use a paper towel to thoroughly and vigorously wash the nebulizer parts in hot, soapy water. Rinse well.

Step #2 Disinfect the nebulizer.

Bring tap water to a boil and boil the nebulizer parts for 10 minutes.

Step #3 Air-dry the nebulizer.

Remove the nebulizer parts from the boiling water and drain on clean surface covered with paper towels. Replace wet paper towels with dry ones, and allow nebulizers to thoroughly air-dry.

Use a clean and disinfected nebulizer for each aerosol treatment. If you wish to reserve the cleaning/disinfecting process for once a day, then have the same number of nebulizer set-ups available as the number of treatments taken per day.

Note: These instructions apply to the PARI nebulizer only.